SILVERBAND Rx ANTIBACTERIAL WOUND DRESSING GEL

Product Description

SILVERBAND Rx Antibacterial Wound Dressing Gel is a water-based gel wound dressing for use in the management of first and second degree burns, wounds such as stasis ulcers, pressure ulcers, diabetic ulcers, lacerations, abrasions, skin tears, surgical incision sites, device insertion site wounds, graft sites and donor sites.

SILVERBAND Rx Antibacterial Wound Dressing Gel contains silver that in laboratory tests has been shown to inhibit the growth of microorganisms such as: Staphylococcus aureus, Pseudomonas aeruginosa, Escherichia coli, antibiotic resistant strains, MRSA and VRE, as well as fungi such as Candida albicans, and Aspergillus niger.

Indications for Use

SILVERBAND Rx Antibacterial Wound Dressing Gel is indicated for the management of:

- First and second degree burns
- Wounds such as: Stasis ulcers, Pressure ulcers, Diabetic ulcers
- Lacerations
- Abrasions
- Skin tears
- Surgical incision sites
- Device insertion site wounds
- Graft sites
- Donor sites

Contents: 32 PPM Proprietary Silver (purified water, nano-silver at 0.01 micron), Propylene Glycol, Triethanolamine, Carbomer.

Directions for Use:

SILVERBAND Rx Antibacterial Wound Dressing Gel should be applied directly to the wound and then covered with a conventional dressing. SILVERBAND Rx Antibacterial Wound Dressing Gel should normally be changed every 24 to 72 hours, but the frequency will depend on the type and condition of the wound.

Precautions: Intended for external use only.

If your condition persists or gets worse, contact your healthcare provider.

Adverse Reactions:

There are no known or reasonably suspected adverse reactions associated with the use of SILVERBAND Rx Antibacterial Wound Dressing Gel. This product is not a medicine.

Caution: This device is restricted for sale under prescription only.

Storage: Store at room temperature.

Manufacturer: ABL Medical, LLC, 705 East 50 South, American Fork, UT 84003, USA

Importer & license holder: NSBT LTD., Me'ir Ya'ari 17, Tel Aviv 6937117

Customer service telephone number: 03-5441042

Approved by the Ministry of Health;

Registration number in the National Medical Device Registry: 27620001

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הרכיבים:
PPM Proprietary Silver (purified water, 32 nano-silver at 0.01 micron), Propylene Glycol, Triethanolamine, Carbomer

תכולה:
32 PPM Proprietary Silver (purified water, nano-silver at 0.01 micron), Propylene Glycol, Triethanolamine, Carbomer.

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